SoClean Adapter für ResMed S9

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SoClean Adapter for ResMed S9

INSTRUCTIONS FOR USE

The adapter for the **ResMed S9** is a special lid with hose attached.

If you need help connecting your adapter, contact SoClean.


EU 00800 5558 8811 | US 866-501-3705 | CA 800-916-4577

soclean.com | info@soclean.com


12 Vose Farm Road | Peterborough, New Hampshire 03458 USA

CONNECTING TO:


SoClean Adapter for ResMed S9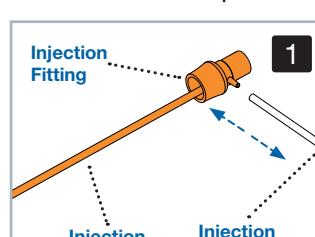


1. Detach the **Injection Fitting** from the **Injection Hose A**. You will no longer need the **Injection Fitting** and **Injection Hose B**. Save these parts in case you need them in the future.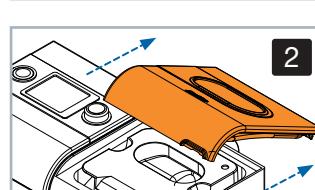


2. Remove the original lid from your humidifier by opening the lid fully, grasping it near the hinges and pushing firmly away from the unit.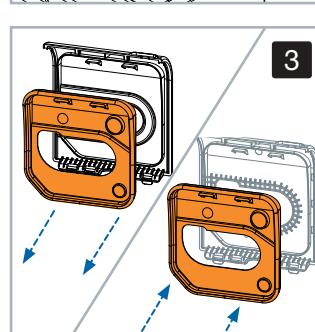


3. Remove the silicone gasket from the original lid and install it onto the new lid, making sure the gasket lies flat and fits over the two tabs. Install the new lid with the black adapter hose by lining up the hinges and pulling firmly toward you until you hear a clicking in both hinges.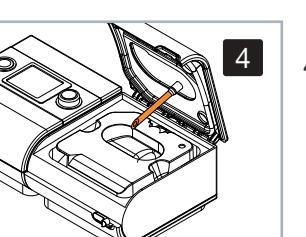

4. Add water to your reservoir, if needed. When closing the new lid, make sure that the hose on the underside passes into the reservoir opening and rests near the bottom.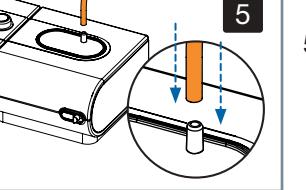


5. Connect the **Injection Hose A** by pushing it onto the end of the port of the adapter lid.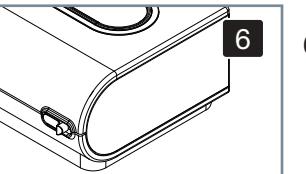


WARNING: **Injection Hose A** must be attached to the SoClean adapter at all times when operating your PAP or the SoClean.

DISCONNECTING THE:


SoClean Adapter for ResMed S9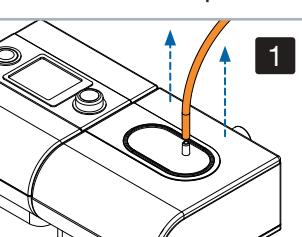


1. Remove the **Injection Hose A** that is connected to the stem of the adapter by firmly pulling it off.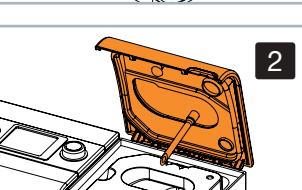


2. Open the adapter lid fully.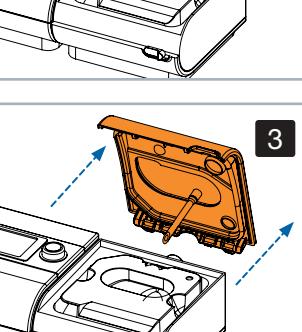

3. Remove the adapter lid by grasping it near the hinges and pushing firmly away from the unit.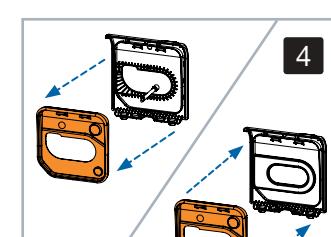

4. Remove the silicone gasket from the adapter lid and install it onto your original reservoir lid making sure the gasket lies flat and fits over the two tabs.

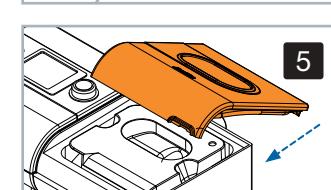5. Install your original reservoir lid by lining up the hinges and pushing firmly toward you until you hear a clicking in both hinges.

6. Close your reservoir lid.

IMPORTANT: If you need to disconnect the SoClean from your adapter, please reinstall the original humidifier lid onto your PAP machine. If you fail to do this, the water in your humidifier will be pushed out through the hose and flooding will result. SoClean is not responsible for damage resulting from equipment mishandling.

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