
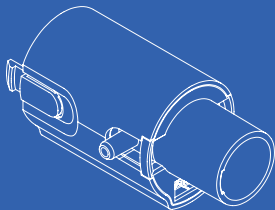
ADAPTER KIT FOR USE WITH PHILIPS DREAMSTATION 2

ADAPTER KIT INCLUDES:

1 - SoClean Express Adapter

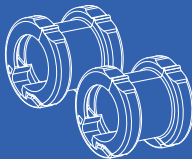2 - Hose Sleeves
[For use with 12 mm hose]

INSTRUCTIONS

PNA1410-DS2-23-CA

Philips Dreamstation 2 Adapter Installation Instructions

**If you need help connecting
your adapter, contact SoClean.**

For SoClean operation information,
please see the User Manual
provided with the device.

An electronic version of the User
Manual is available at [SoClean.com](https://www.soclean.com)
or you may contact Customer Care.

US: 1-866-501-3705 | CA: 1-800-916-4577
[SoClean.com](https://www.soclean.com) | info@soclean.com

SoClean Express Adapter
[Connects to the Hose Port]

SoClean, is an independent company not affiliated with Philips Respironics. Names and associated trademarks are solely owned by the respective companies and manufacturers. Depictions of brand logos on our website are for informational and educational purposes only and are used for customer sleep equipment recognition. We are not affiliated or endorsed by any sleep equipment manufacturer.

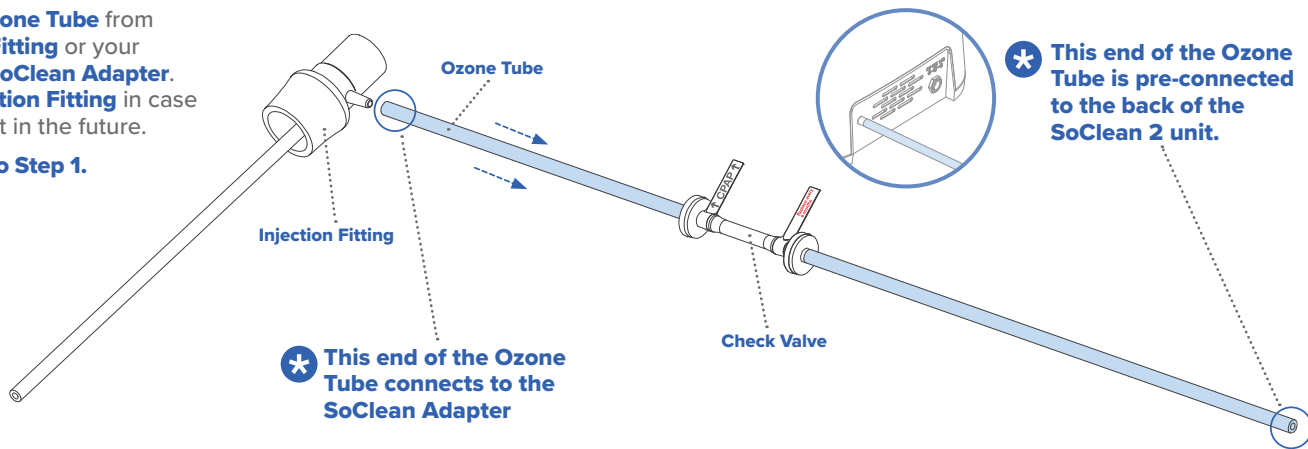
If you are a **SoClean 3** user, move on to **Step 1**.

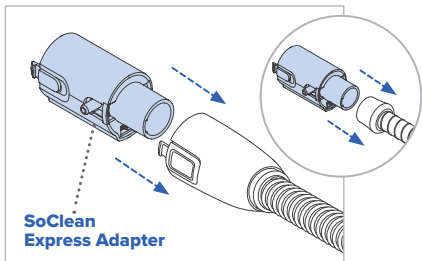
If you are a **SoClean 2** user:

Detach **Ozone Tube** from **Injection Fitting** or your previous **SoClean Adapter**. Save **Injection Fitting** in case you need it in the future.


Move on to Step 1.


1

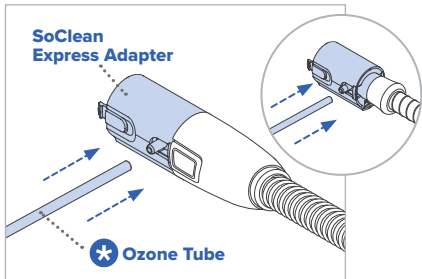Connect the **SoClean Express Adapter** to the heated or non-heated hose.

2

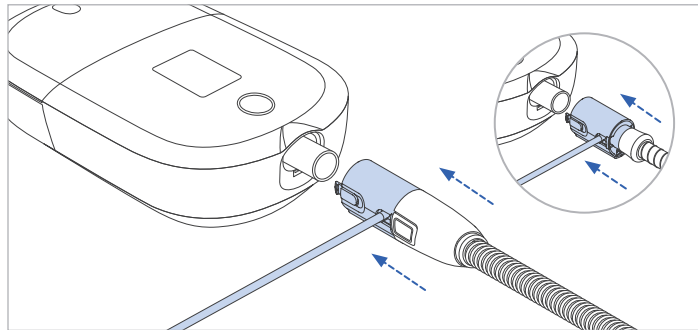Connect the **Ozone Tube** to the **SoClean Express Adapter**.

3

Connect the **SoClean Express Adapter** with hose and **Ozone Tube** to the DreamStation 2.

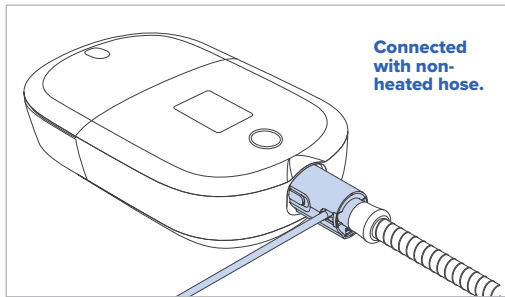
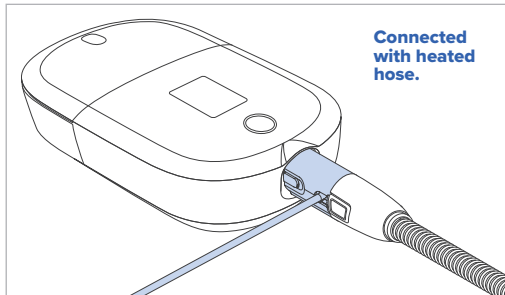
WARNING: The **Ozone Tube** should always be connected to the **SoClean Adapter** and the back of the SoClean unit when operating the SoClean.

WARNING: Ensure sleep equipment hose and **SoClean Adapter** are connected to the sleep equipment machine prior to operating a SoClean Cycle.

You are now ready to use the SoClean.

NOTE: The SoClean Adapter can be left connected to your sleep equipment machine after initial setup. You do not need to disconnect the SoClean Adapter to operate either the sleep equipment machine or the SoClean 3.

Adding distilled water to your humidifier reservoir:

When you need to add water to your reservoir, remove the reservoir from the sleep equipment machine. Add distilled water, and reinsert into the sleep equipment machine.

NOTE: SoClean is not responsible for damage to your sleep equipment machine or personal property resulting from equipment mishandling.

IMPORTANT: For travel, remove the SoClean Express Adapter from your sleep equipment and reconnect your Hose to your sleep equipment. If you only disconnect the Ozone Tube from your Express Adapter, the water in your humidifier will be pushed out through the Adapter Ozone Port during sleep equipment use and an overflow will result.

SoClean Hose Sleeve Installation Instructions

SoClean Hose Sleeve
[For use with 12 mm hose]

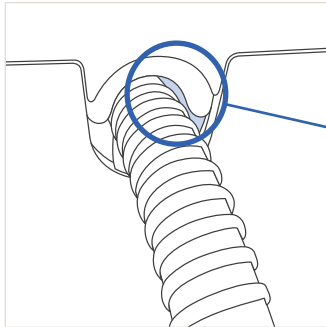
When to use the Hose Sleeve:

**The Hose Sleeve is required if you use a 12 mm hose.
It is not compatible with 15 mm or larger diameter hoses.**


An extra sleeve is included in the event of sleeve breakage.
You will only use one Hose Sleeve at a time.

The SoClean requires a snug fit around the hose when placed into the Hose Slot and the lid is closed. If there is space around the hose and this is not achieved, the SoClean will not operate.

If there is visible space around the hose when placed in the Hose Slot, you may be using a 12 mm hose and require the Hose Sleeve be installed.


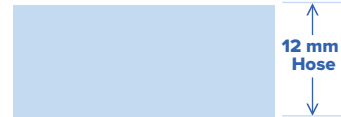
Determining your hose size:

If you determine that you have a 15 mm hose and the SoClean is still not operating, contact Customer Care.


How to use the Hose Sleeve:

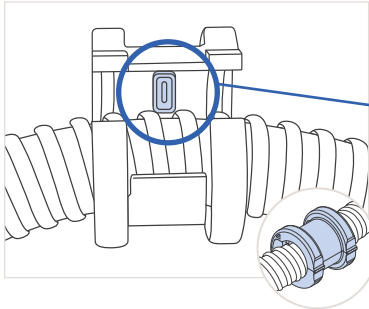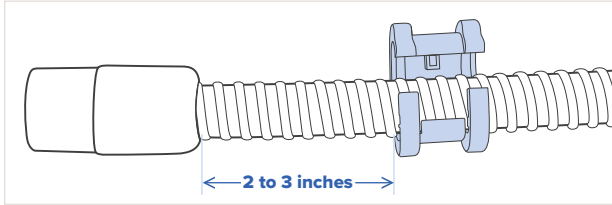- 1** Position the Hose Sleeve 2 to 3 inches from the connector.


- 2** Position the hose on the Nub side of the Hose Sleeve. Slide the hose until the Sleeve "clicks" into place between two of the hose ribs and close the Sleeve.

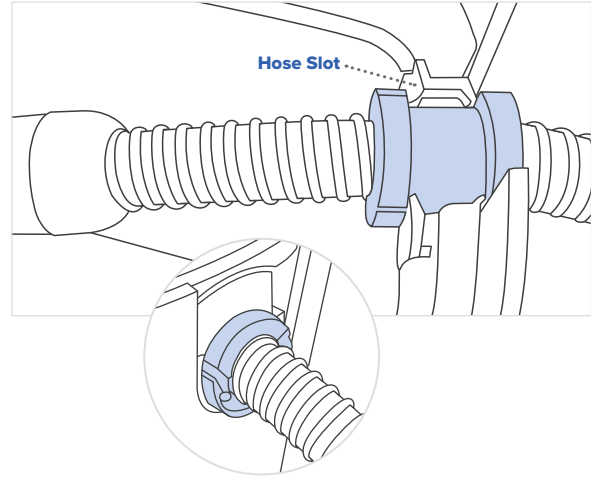
- 3** Line up the Hose Sleeve with the gasket, place it in the Hose Slot and shut the lid.


ADAPTER DISCONNECTION:

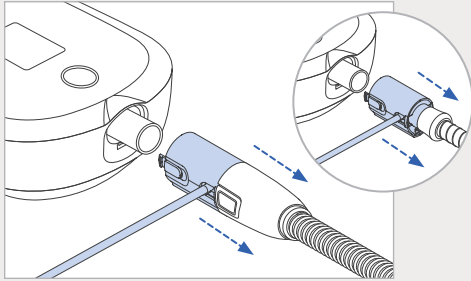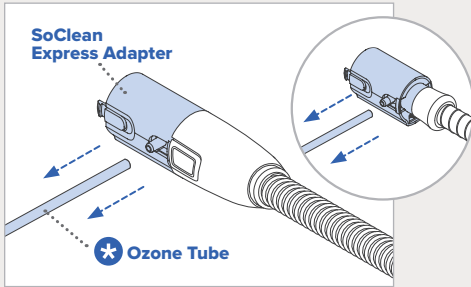
1
Disconnect the **SoClean Express Adapter** with hose and **Ozone Tube** from the DreamStation 2.

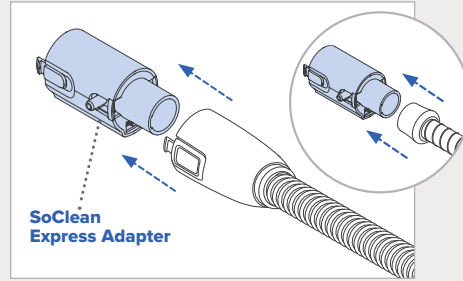2
Disconnect the **Ozone Tube** from the **SoClean Express Adapter**.

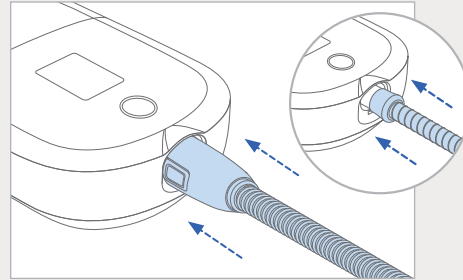3
Disconnect the **SoClean Express Adapter** from the heated or non-heated hose.

4
Connect heated or non-heated hose to DreamStation 2.
You are now ready to use your sleep equipment.

Using Your SoClean

The below diagram shows a SoClean 3 connected.
The SoClean 2 setup will look similar but include the check valve.

SoClean 3 connected to Philips DreamStation 2

Once you have completed the setup steps, you are ready to begin the daily maintenance of your sleep equipment accessories with your SoClean.

Safety Checklist Before Using Your SoClean

- Sleep equipment accessories are washed in sink before first use
- SoClean Adapter is installed on sleep equipment machine
- Ozone tube is connected to sleep equipment machine
- Sleep equipment hose is still connected to sleep equipment machine and mask
- SoClean is powered on
- Filter is installed

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SoClean Inc.