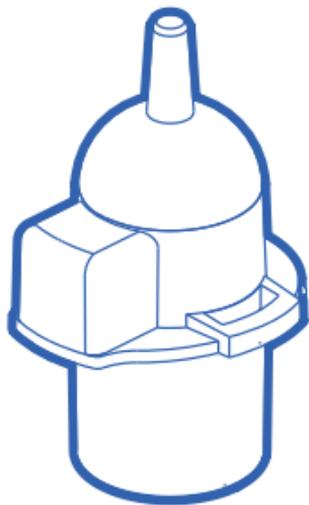
ADAPTER FOR HOSE AND MASK MAINTENANCE

Compatible With Most PAP Hoses

INSTRUCTIONS FOR USE
PNA1604-CA

If you need help connecting your adapter, contact SoClean.


For SoClean operation information, please see the User Manual provided with the device.

An electronic version of the User Manual is available at SoClean.com or you may contact Customer Care.


IMPORTANT: This adapter needs to fit snugly. If the fit appears too tight, do not force adapter into tubing. If the adapter is loose, call Customer Care.

US: 1-866-501-3705 | **CA:** 1-800-916-4577
SoClean.com | info@soclean.com

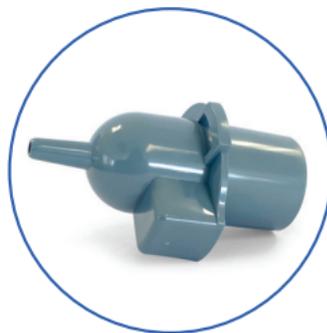
Installation Instructions


SoClean Adapter

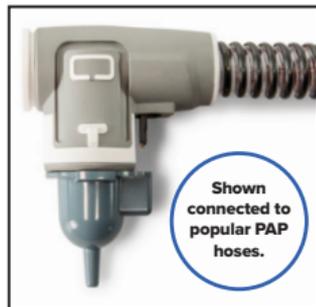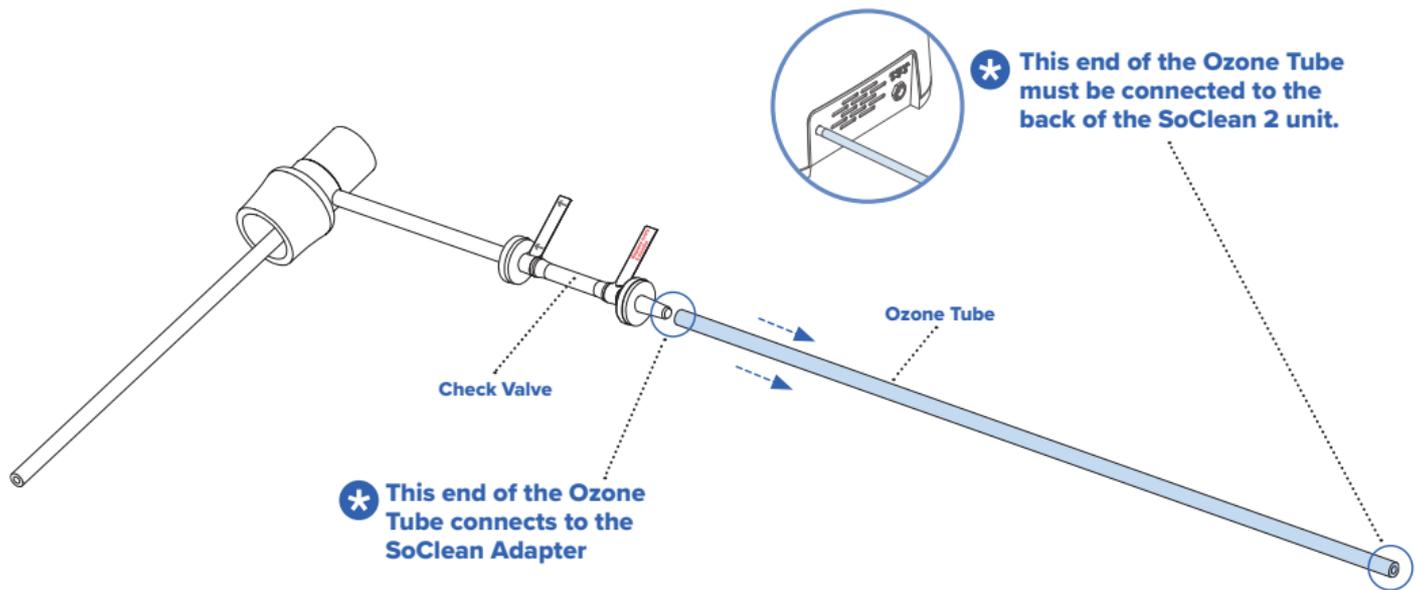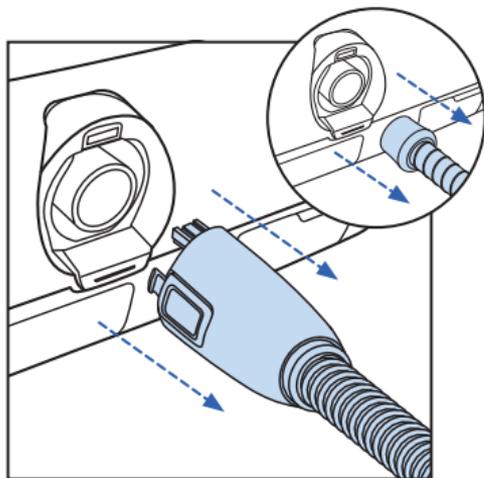
Detach the **Ozone Tube** from the **Check Valve** or your previous **SoClean Adapter**.


1

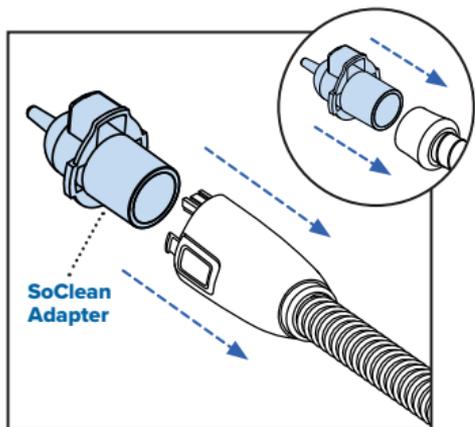Remove the hose from your PAP device.

2

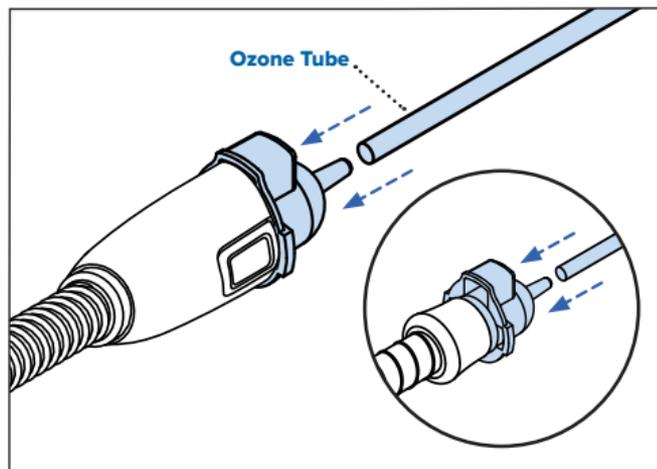Install the **SoClean Adapter** onto the hose.

3

Connect **Ozone Tube** to the **SoClean Adapter**. You are now ready to use your SoClean.

Place mask into the **SoClean Chamber** and run the maintenance cycle.

WARNING: The **Ozone Tube** must always be connected to the **SoClean Adapter** and the back of the SoClean unit when operating the SoClean.

Once you have completed the setup steps, you are ready to begin the daily maintenance of your PAP hose and mask with your SoClean.

Safety Checklist Before Using Your SoClean

- PAP hose and mask are washed in sink and dried before first use
- SoClean Adapter is installed on PAP hose
- Ozone Tube is connected to the back of the SoClean unit and the SoClean Adapter
- PAP hose is still connected to the mask
- SoClean is powered on
- Filter is installed


Using Your SoClean

The below image shows your SoClean 2 connected.


Reconnecting Your Sleep Equipment

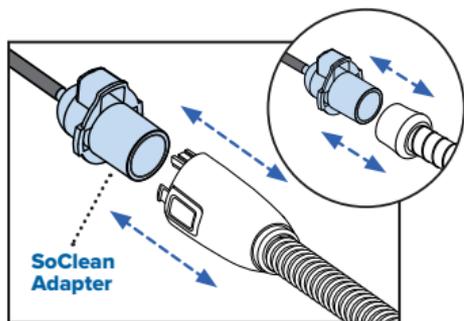Once the **Maintenance Cycle** is complete, wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

1

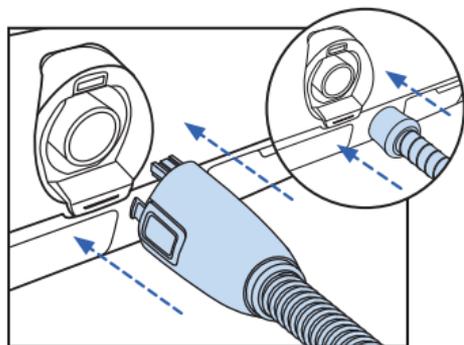Remove the **SoClean Adapter** from your hose.

2

Reconnect your hose to the PAP device hose port.

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SoClean Inc.