SoClean®

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INSTRUCTIONS FOR USE
INSTRUCTIONS D'UTILISATION

ENGLISH

Use the Neutralizing Pre-Wash to hand wash your sleep equipment accessories in the sink in order to remove residue, like oil from your skin, that can naturally build up over time.

The Neutralizing Pre-Wash is a mild, fragrance-free, hypoallergenic washing detergent and may be used for periodic cleaning as recommended by the sleep equipment manufacturer.

Pre-wash sleep equipment

Disconnect your mask, hose and reservoir from your sleep equipment. Before you disconnect your sleep equipment to wash, look at where your breathing hose is connected. You will reconnect either the breathing hose or the injection fitting to the same location, depending on your SoClean device and set-up instructions.

CUSTOMER CARE
US 866-501-3705 CA 800-916-4577

4 liters (1 gallon)

1

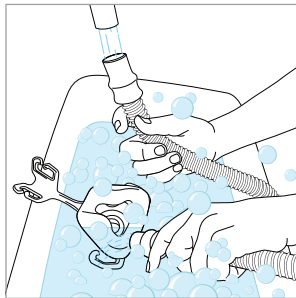
Mix one capful of the Neutralizing Pre-Wash with four liters (1 gallon) of water in a suitable clean container.

2

Wash your mask, hose and reservoir according to manufacturer instructions.

3

Rinse thoroughly with clean water and let dry.

If you have a reservoir, reinsert the reservoir into the sleep equipment. Before reattaching the breathing hose, see the set-up instructions in the SoClean device user manual. If your sleep equipment requires an adapter, see adapter instructions for set-up.

WARNING: Do not put Neutralizing Pre-Wash or water into the SoClean Chamber. Do not use the Neutralizing Pre-Wash on or in the SoClean 3 device. The Pre-Wash is intended for washing sleep equipment accessories only.

CAUTION: Use the Neutralizing Pre-Wash or unscented mild detergent to wash your sleep equipment accessories according to the sleep equipment manufacturer's recommended method and interval. Use the SoClean 3 for daily supplemental maintenance of your sleep equipment accessories.

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SoClean, Inc.