User Manual

Supplemental PAP Hose and Mask
Maintenance Device

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The SoClean 2 is a Supplemental PAP Hose and Mask Maintenance Device that is intended for use by a single-user, for personal at-home-use.

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ENGLISH

Thank You!

Thank you for purchasing the SoClean 2 Supplemental PAP Hose and Mask Maintenance Device.*

This manual contains a product overview and key set-up instructions.

The SoClean patented process kills 99.9% of microorganisms* commonly found on PAP masks and hoses. SoClean recommends users follow CPAP-manufacturer’s instructions for cleaning their PAP accessories, and that the SoClean 2 is intended for use as a supplement to that cleaning.

SoClean is committed to developing innovative and safe ways to improve the health and well-being of PAP users.

Please feel free to contact us with any questions you may have at SoClean.com/contact.

*See Indication of use, Cautions and Specifications.

TABLE OF CONTENTS

INDICATIONS FOR USE, CONTRAINDICATIONS, SPECIFICATIONS, WARNINGS & CAUTIONS.....	6
YOUR HEALTH AND OZONE SAFETY.....	8
CLASSIFICATION, MANUFACTURER, SYMBOLS & FCC NOTICE	9
INCLUDED WITH YOUR SOCLEAN 2.....	10
SOCLEAN 2 CONTROL PANEL.....	14
WASH, RINSE AND DRY YOUR PAP HOSE AND MASK.....	16
USING THE HOSE AND MASK ADAPTER WITH YOUR SOCLEAN 2	17
CONNECT POWER.....	20
SET LANGUAGE	22
SET CLOCK & CYCLE SCHEDULE	23
USING YOUR SOCLEAN 2.....	26
RECONNECTING YOUR PAP EQUIPMENT	28
CARE & MAINTENANCE OF YOUR SOCLEAN 2.....	29
FAQ.....	30

SoClean.2

INDICATIONS FOR USE, CONTRAINDICATIONS, SPECIFICATIONS, WARNINGS AND CAUTIONS

Indications for use: The SoClean 2 is a Supplemental PAP Hose and Mask Maintenance* Device that is intended for use by a single-user, for personal at-home-use to maintain PAP hoses and masks. SoClean 2 kills 99.9% of microorganisms* commonly found on PAP hoses and masks when set at the 12-minute cycle time duration.

SoClean recommends users follow PAP-manufacturer's instructions for cleaning their PAP hoses and masks. The SoClean 2 is intended for use as a supplement to that cleaning.

Contraindications for Use: Persons with underlying lung diseases, such as asthma and chronic obstructive pulmonary disease (also known as COPD, which includes emphysema and chronic bronchitis), and those with cardiovascular disease may be sensitive to ozone and should consult with their health care professional before using this product. Safety in pregnant or breastfeeding women and children under the age of 22 have not been established. Consult with your health care professional before using this product.

Specifications:

Electrical characteristics:

AC adapter input: AC 100~240V, 50/60HZ, 0.5A

AC adapter output: DC 12V, 1.5A max.

Power consumption: 18W max.

Environmental characteristics:

Operating: 10°C to 38°C (50°F to 100°F), 15% to 70% humidity

Storage and transport conditions:

-20°C to +55°C (-4°F to 131°F), 15% to 70% humidity

Ozone concentration:

When operated in a 11.6 m² room with 2.4 metre-high ceilings, the average ambient ozone concentration in the room over the operating cycle is < 0.05 parts per million (PPM).†

Ozone Cycle Run Time Duration

12 minutes for maintenance

Physical characteristics:

Dimensions: 200 x 184 x 225 mm

Weight: 2.5 kg

Cord length: 142 cm

†Data on file.

Read this instruction manual before operating this device. Your SoClean 2 should be used only according to the procedures described in this manual. If it is used in a manner not specified by SoClean®, the safety protection provided by the equipment may be impaired.

*U.S. independent lab testing with SoClean 2 demonstrates 99.9% reduction of the following microorganisms comprised of the following bacteria: *Staphylococcus haemolyticus*, *Staphylococcus hominis*, *Staphylococcus epidermidis*, and yeast: *Candida albicans*

**The term "maintenance" herein refers to SoClean 2 performance killing 99.9% of microorganisms* commonly found on PAP masks and hoses when set at the 12-minute cycle time duration.

! WARNING: Do not use highly fragrant soaps to wash the PAP hose and mask when using the SoClean 2.

! WARNING: Discontinue use if acne or a rash develops along the mask contact line and call SoClean customer care.

! WARNING: Ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Pay particular attention to the AM/PM setting if using 12-hr mode. Failure to do so may result in the SoClean 2 operating at an unexpected time.

! WARNING: In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time.

! WARNING: Do not physically alter the Hose Detect Switches on the SoClean 2. Regularly inspect your SoClean 2 for physical damage, such as cracks or broken structure, damaged gaskets, and cracks in the sleep equipment hose. Discontinue use if damage is detected and contact SoClean for assistance.

! WARNING: Keep away from water sources including rain, bathtubs, sinks and pools. To clean your SoClean 2, wipe with a damp cloth. Do not submerge the SoClean 2 device in liquid or use chemical cleaners.

! WARNING: Keep away from children. Do not place any living thing inside the Chamber during use.

! WARNING: Do not inhale from the Ozone Tube or the Ozone Tube outlet on the back of the SoClean 2.

! WARNING: Only rest the flexible section of the PAP Hose, not the rigid connector, in the Hose Detect Slot.

! WARNING: Do not breathe through your PAP mask if the SoClean 2 is operating a cycle.

! WARNING: Do not use in explosive air environments, near gas vapours or other flammables, etc. Do not place any flammable or ignitable items on top of the SoClean 2.

! WARNING: Keep the SoClean Lid closed until the green status light illuminates. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

! WARNING: Do not remove gasket from the Hose Detect Switch Slot.

! CAUTION: The Lid Gasket is removable. Ensure that the Lid Gasket is installed properly before operating a cycle.

! WARNING: Do not dismantle the SoClean 2.

! CAUTION: Please be sure the black Ozone Tube from the back of the SoClean 2 is connected to the SoClean Adapter before operating the SoClean 2.

! WARNING: When operating your SoClean 2 per the instructions for use, if there is a noticeable scent of ozone or other unfamiliar odor or you experience headache, cough, or shortness of breath, leave the room for 4 hours. Contact SoClean Customer Care before running your machine again.

! WARNING: Do not route your PAP Hose through both Hose Detect Slots while or after using your PAP. This may result in the SoClean 2 device running at an unexpected time.

! WARNING: Remove clear outer wrapper and blue tape from Cartridge Filter.

YOUR HEALTH AND OZONE SAFETY

- Ozone, the triatomic form of oxygen (O₃), is known as activated oxygen.
- Ozone is an effective, invisible disinfectant.
- The SoClean 2 uses ozone to maintain Positive Airway Pressure (PAP) masks and hoses; ozone contacts the PAP masks and hoses only, not the user.
- The SoClean 2 is designed to produce only enough ozone for maintenance of your PAP mask and hose.
- While breathing large amounts of ozone can irritate breathing passages of humans, following the instructions for use ensures that SoClean 2 users will not breathe in large amounts of ozone.
- Ozone quickly dissipates in the atmosphere. If you accidentally interrupt the cycle and release ozone by opening the SoClean Chamber before the end of the cycle, simply step away from the SoClean **for a minimum of 6 hours** to avoid unintended exposure.
- Ozone is a gaseous disinfectant that naturally breaks down after it is generated. The SoClean is designed to safely contain ozone within the system and has a catalyst Cartridge Filter inside the SoClean Chamber to convert ozone back to oxygen.

Contraindications for Use:

Persons with underlying lung diseases, such as asthma and chronic obstructive pulmonary disease (also known as COPD, which includes emphysema and chronic bronchitis), and those with cardiovascular disease may be sensitive to ozone and should consult with their health care professional before using this product. Safety in pregnant or breastfeeding women and children under the age of 22 have not been established. Consult with your health care professional before using this product.

- **WARNING:** When operating your SoClean 2 per the instructions for use, if there is a noticeable scent of ozone or other unfamiliar odor or you experience headache, cough, or shortness of breath, leave the room for 4 hours. Contact SoClean Customer Care before running your machine again.
- **WARNING:** If you are experiencing headaches/migraines, or symptoms such as cough or discomfort in your nose or throat discontinue use and call your health care provider.
- **WARNING:** Discontinue use and call your health care provider if you experience any cardiopulmonary symptoms.


- **WARNING:** Do not physically alter the Hose Detect Switches on the SoClean 2. Regularly inspect your SoClean 2 for physical damage, such as cracks or broken structure, damaged gaskets, and cracks in the sleep equipment hose. Discontinue use if damage is detected and contact SoClean for assistance.
- **CAUTION:** Discontinue use of your SoClean 2, consult your health care provider and contact SoClean Customer Care if acne or a rash develops along the mask contact line.

CLASSIFICATION, MANUFACTURER, SYMBOLS & FCC NOTICE

Classification

Health Canada Device License Number 100197, Device Classification: Class 2

Symbols

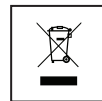	Federal Communications Commission		Manufacturer SoClean, Inc. 1 Vose Farm Road Peterborough, New Hampshire 03458 U.S.A.
	Consult manual		Waste Electrical and Electronic Equipment (WEEE). It is the responsibility of the end user to dispose of this equipment at a designated collection point.
	Catalogue number		Not made with natural rubber latex
			Date of manufacture
			Do not use if package is damaged.

FCC Notice


Please note that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 18 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

INCLUDED WITH YOUR SOCLEAN 2

Number	Part name	SKU	Description
1	Removable Hose Slot Plug	PN1214	This allows you to set up the SoClean 2 device on either side. Place this plug in the Hose Slot that you don't use.
2	SoClean Chamber	N/A	Place your mask in here when you run a Maintenance Cycle.
3	Lid Gasket	N/A	Ensures a tight seal around the SoClean Chamber when the SoClean 2 Lid is closed. Do not close the Lid without this part in place.
4	SoClean Lid	N/A	Close the Lid after you place your mask in the SoClean Chamber to ensure that the cycle occurs.
5	Cartridge Filter Kit	PN1207-CA	This Cartridge Filter converts ozone back into normal oxygen. Remove outer plastic wrapper and blue tape prior to installation.
6	Hose Detect Switch	N/A	The Hose Detect Switch detects when the SoClean 2 is configured for operation by detecting the presence of the hose in the Hose Slot when the Lid is closed. If no hose is detected, the SoClean 2 will not run a cycle. Hose Detect Switch operation can be inspected by gently pushing down and releasing the grey plastic switch and hearing an audible click. The switch should move freely down and up when pressed gently.

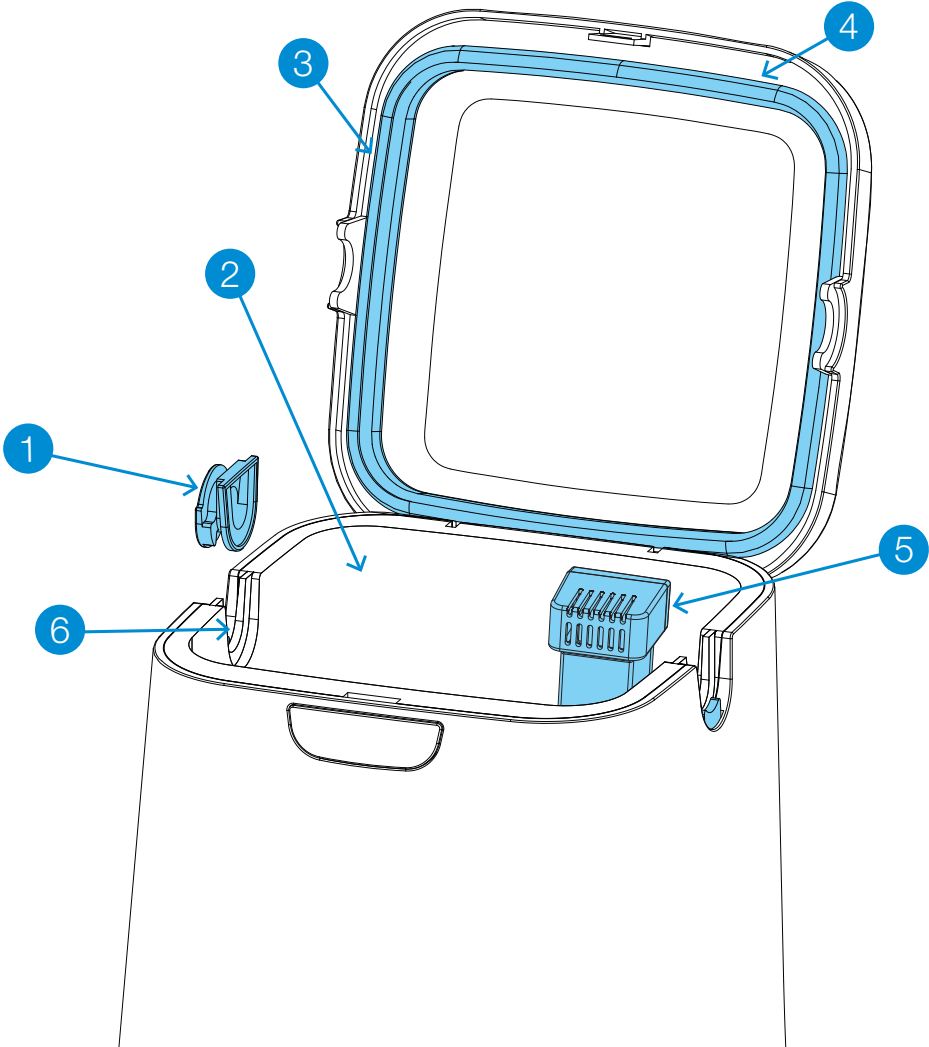INCLUDED WITH YOUR SOCLEAN 2




Number	Part name	SKU	Description
7	Control panel	N/A	Buttons, display and cycle light indicator for SoClean functions.
8	Ozone Tube A	N/A	You will only be using Ozone Tube A, the longest section of black tubing which carries ozone from the SoClean 2 through the PAP hose and mask.
9	Check Valve	N/A	The Check Valve is not required for use with the Hose and Mask Adapter
10	Injection Fitting	N/A	The Injection Fitting is not required for use with the Hose and Mask Adapter
11	Power Supply	PN1222	Supplies power to your SoClean 2.

Replacement Parts


A message appears on your SoClean 2 after a certain amount of use (typically around six months) as a reminder to replace the Cartridge Filter.

To reorder supplies, contact your authorized reseller or visit **SoClean.com**.

Follow your PAP manufacturer's instructions for cleaning your PAP hose and mask with mild, unscented soap and warm, drinking-quality water.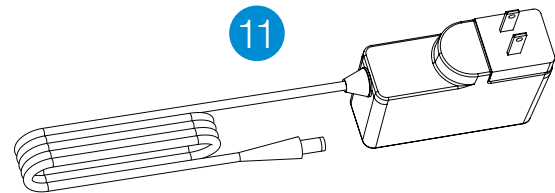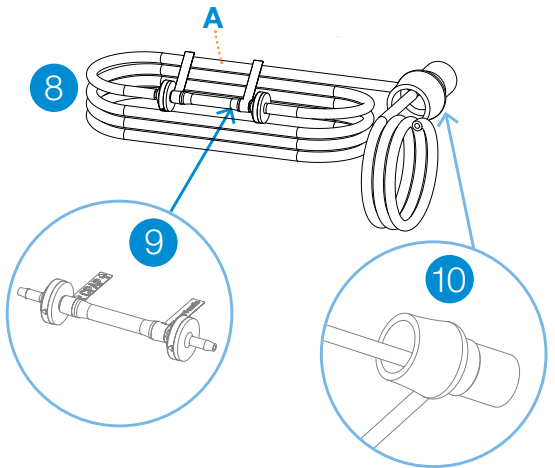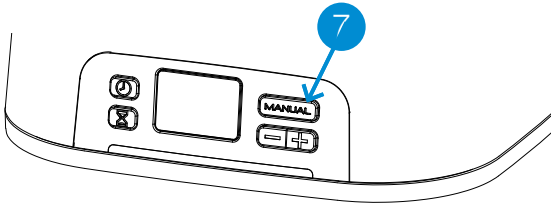

Replacement Parts

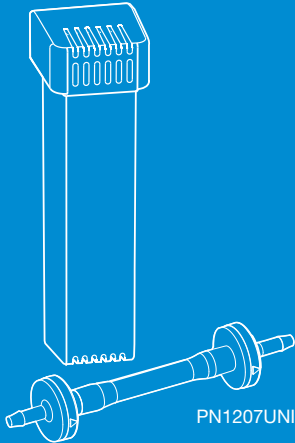
The Check Valve is not required for use with the Hose and Mask Adapter.

WARNING: Please purchase your Cartridge Filter Kits from SoClean® only. Authentic Cartridge Filter Kits can be purchased from SoClean via multiple online marketplaces, such as Amazon®, in addition to SoClean.com. However, please confirm that the seller of your Cartridge Filter Kit is SoClean, and not any other seller, to ensure the authenticity of your Cartridge Filter Kit. SoClean has discovered counterfeit Cartridge Filter Kits being sold, particularly on Amazon® and eBay®, with Check Valves and Filters that leak and can cause damage to your SoClean machine or to your property due to water leakage from the counterfeit Check Valves. Please contact SoClean immediately if you have had any issues with your Cartridge Filter Kit. Amazon® is a registered trademark owned by Amazon Technologies, Inc. eBay® is a registered trademark owned by eBay, Inc. SoClean® is a registered trademark owned by SoClean, Inc.

SOCLEAN 2 CONTROL PANEL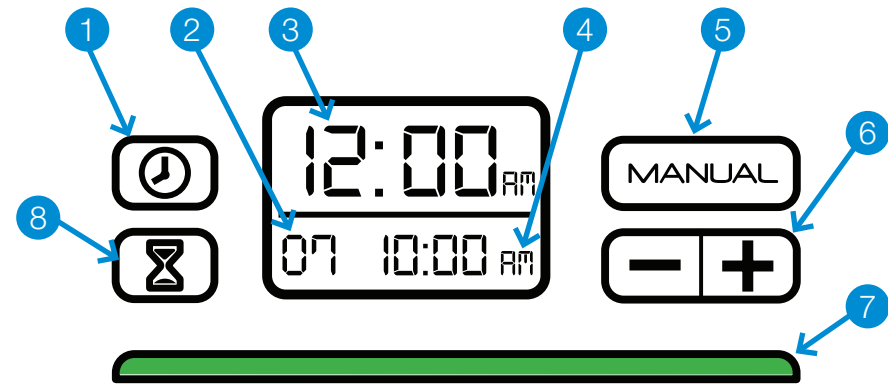

⚠ WARNING: In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time.

Number	Button(s)/Display	Description
1		Sets the clock.
2	Cycle run time (duration)	The duration, in minutes, of the cycle. The default Cycle Run Time is set to 7 minutes. A duration of 12 minutes is required to achieve maintenance.
3	Current time	The current time. Default time is set to 12:00 a.m.
4	Daily start time	The time of day when the cycle starts. The default daily cycle start time is set to 10:00 a.m.
5		Starts a cycle immediately. The scheduled cycle will still occur.
6		Decreases (-) or increases (+) time.


Number	Button(s)/Display	Description
7	Status light	The light below the display indicates the cycle status. A green light indicates that the cycle is complete, and you can remove your PAP mask from the SoClean Chamber. RED – The SoClean 2 is operating and generating ozone. YELLOW – The 2-hour cycle is still active. The ozone is working and dissipating. GREEN – Cycle is complete. The light turns off when you open the Lid. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.
8		Sets daily start time.
	and	Changes clock between the 24-hour or 12-hour format.

Messages

The SoClean 2 display panel shows messages that may require action. This list shows the messages and how to correct them:

Message	What to Do
Hose not present. Cycle will not run.	Ensure that the PAP Hose is seated securely in the Hose Slot. Check that the Removable Hose Slot Plug is seated correctly on the other side.
Order Filter Kit.	Your Cartridge Filter has reached the end of its life based on your use.
Cycle in process. Do not open the Lid!	Ozone present. The SoClean 2 is performing a Maintenance Cycle in a well ventilated room. While the SoClean 2 Maintenance Cycle is in progress, leave the room. The green Cycle indicator light means it is complete. You can now open the Lid. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

WASH, RINSE AND DRY YOUR PAP HOSE AND MASK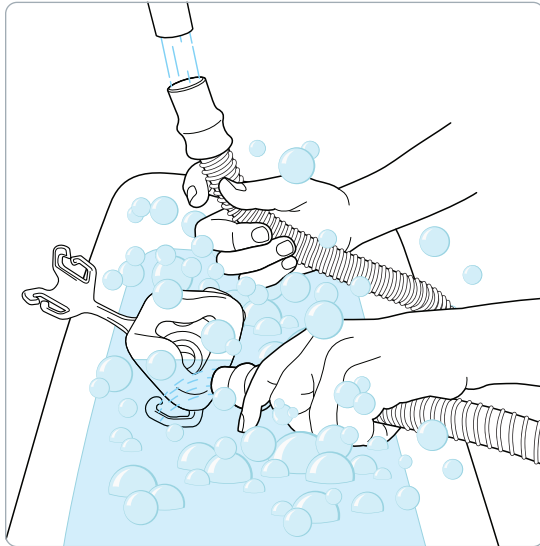


Wash, rinse, and dry your hose and mask as recommended by the manufacturer.

Your SoClean 2 should be used as a supplement to your cleaning routine, not a replacement.

Washing removes oils, films, debris and other build up that can naturally form on your mask and hose from use.



USING THE HOSE AND MASK ADAPTER WITH YOUR SOCLEAN 2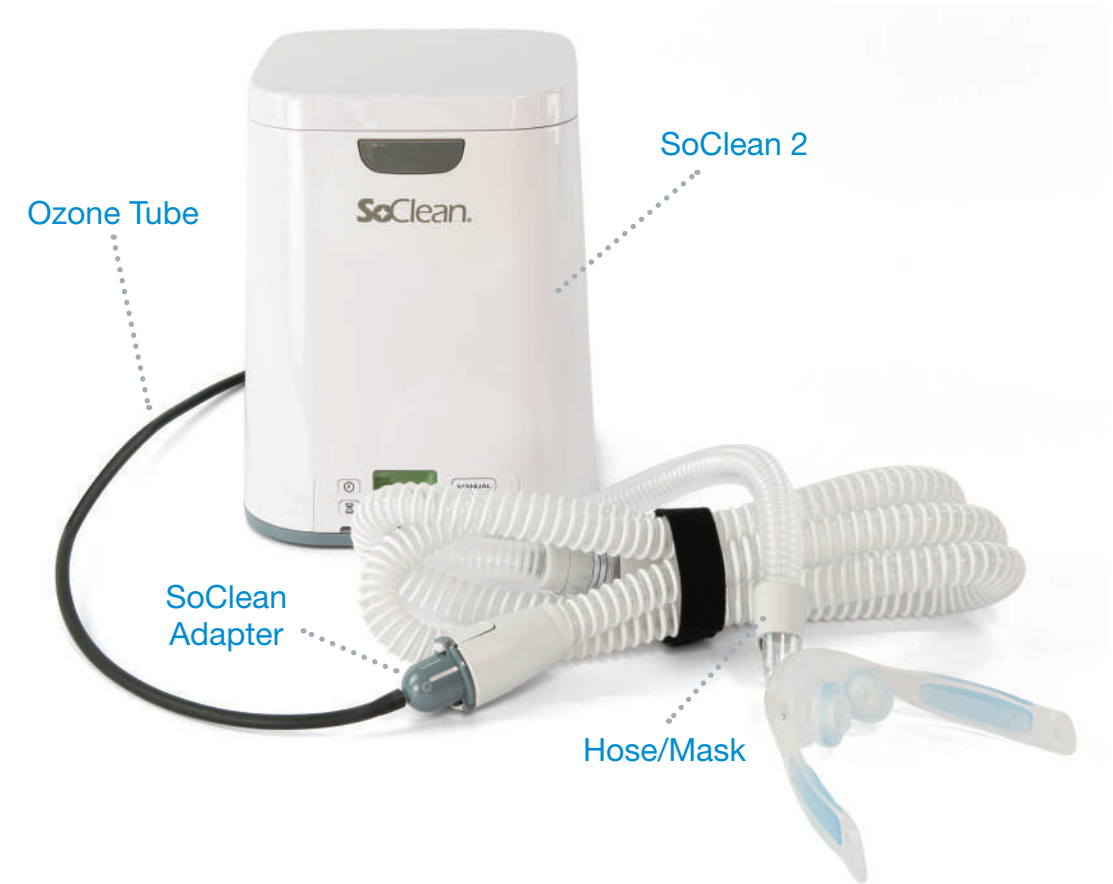

To use the Hose and Mask Adapter with your SoClean 2:

Detach **Ozone Tube A** from the included **Check Valve** or your previous **SoClean Adapter**.




Move on to Step 1.

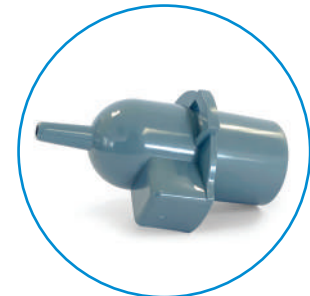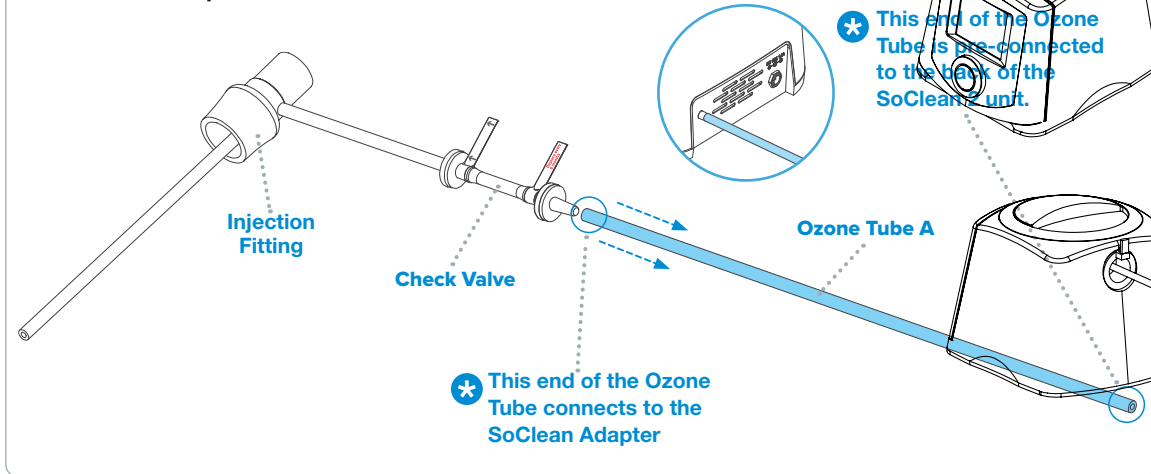
SoClean Adapter


IMPORTANT: This adapter needs to fit snugly.
If the fit appears too tight, do not force adapter into tubing.
If the adapter is loose, call Customer Care at 800-916-4577.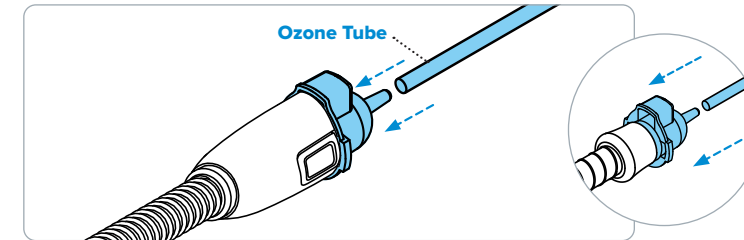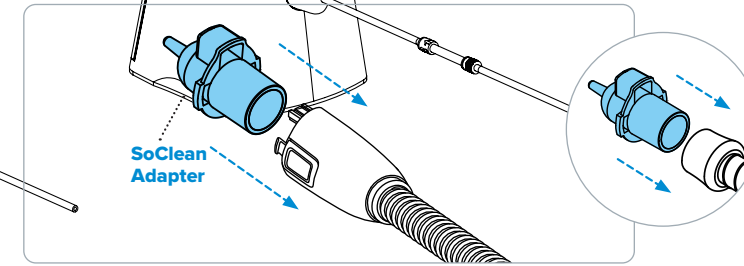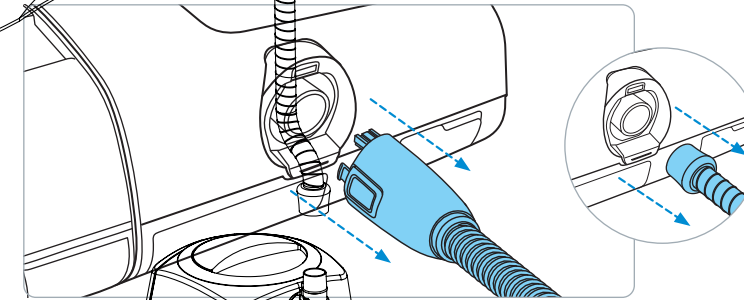

WARNING: The Ozone Tube must always be connected to the SoClean Adapter and the back of the SoClean 2 unit when operating the SoClean 2.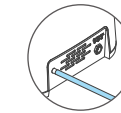

CONNECT POWER

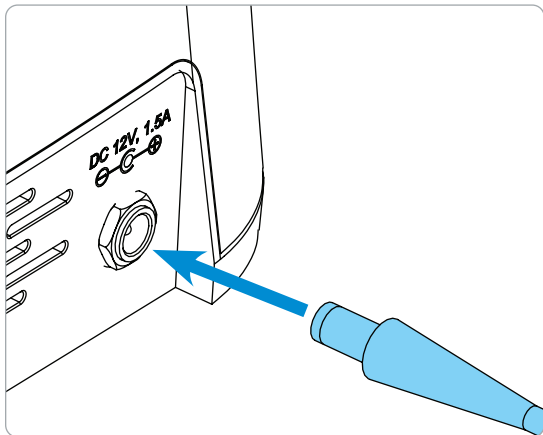
Plug the Power Supply into the connector marked DC 12V, 1.5A on the back of the SoClean 2. The Power Supply should be plugged into a wall outlet that is easy to access and observe.

⚠ WARNING: In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time.

NEW CONNECTION OPTION

PAP Bypass	Model	PN
Connect hose and mask only	Compatible with most PAP hoses	PNA1604-23-CA

SET LANGUAGE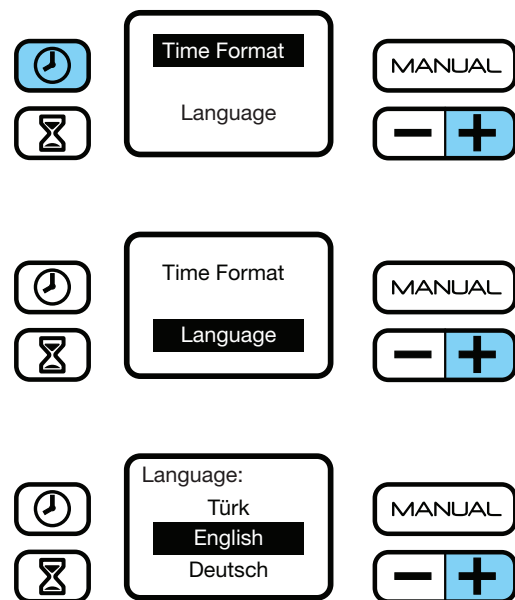

Language

To choose the correct language:

1. Press the **+** and **⌚** buttons simultaneously.
2. Choose 'Language' by using the **+** button.
3. With 'Language' selected, press the **MANUAL** button.
4. Scroll through language options by using the **+** button.
5. Press the **MANUAL** button to select your language of choice.
6. Press the **⌚** button to save.


SET CLOCK AND CYCLE SCHEDULE

WARNING: Ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Pay particular attention to the AM/PM setting if using 12-hr mode. Failure to do so may result in the SoClean 2 operating at an unexpected time.

WARNING: In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time.

If you unplug the SoClean 2 or experience a power loss, you will need to reset the current time only. The settings will remain saved. For safe use, operate your SoClean 2 only as instructed in this user manual.

The SoClean 2 comes preset to start a 7-minute cycle at 10:00 a.m. daily. However, **you must set the clock to your current time**. The following procedures describe how to set the clock, schedule and start a cycle immediately.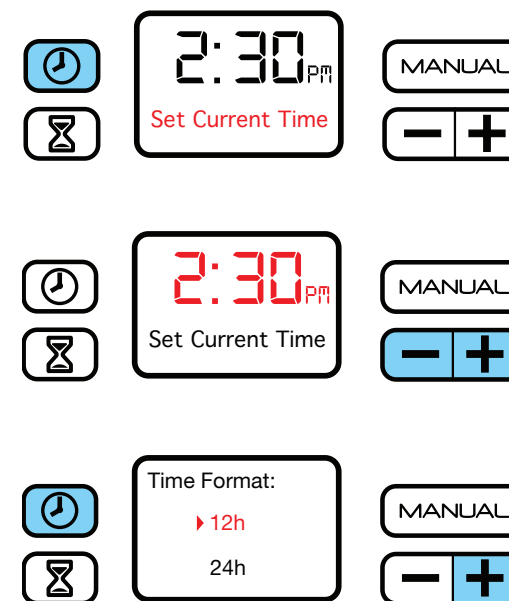

Set the clock/current time



1. Press the **⌚** button.
2. Change the time by pushing either the **-** or **+** button until your current time is reached. By holding down the button, the time moves faster. Press the **⌚** button once more to store your time and return to the home screen.

Set time format

Default is 12-hour format.

To display in 24-hour format (military time):

1. Press the **+** and **⌚** buttons simultaneously.
2. With 'Time Format' selected, press the **MANUAL** button.
3. Choose either the 12- or 24-hour format by using the **+** button.
4. Press the **MANUAL** button to select preferred setting.
5. Press the **⌚** button to save.

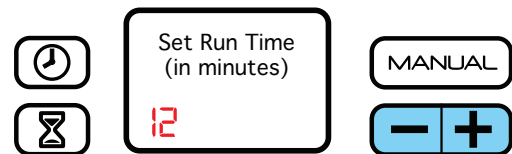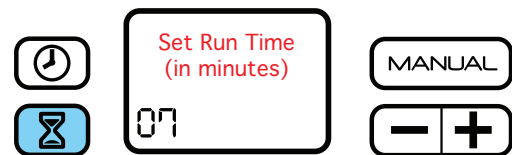Cycle Settings

Set cycle run time (duration) and the daily start time

These two processes follow each other in this set-up.

1. Press and release the  button to display the menu pictured on the left.
2. Press and release the  or  button to decrease or increase the number of minutes of your cycle run time.

Note: The factory set cycle run time is 7 minutes. A duration of 12 minutes is required to achieve maintenance. Press and release the  button to increase the cycle run time to 12 minutes.

3. Press and release the  button again to save your cycle run time and schedule the daily start time. To skip the daily start time set-up, press the  button again when you reach this step.

Note: The default daily start time is 10:00 a.m.

4. Press and release the  or  buttons to reach the desired daily start time. Press the  button again to save and return to the home screen.

Note: Your SoClean 2 should be operated in a well ventilated room. The entire process requires 2 hours plus the Cycle run time to complete. Leave the room during the Maintenance Cycle. A green indicator light means that the Cycle has completed. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

Remove the Removable Slot Plug and place inside the SoClean Chamber.

Manual mode


Ensure the SoClean 2 is being operated in a well ventilated room and that your mask and hose are in place and the SoClean Lid is closed.

1. Press and release the  button to begin the Cycle.

The Maintenance Cycle starts immediately and requires 2 hours plus the Cycle run time to complete. Leave the room during this time. Wait for the green light before removing your mask. It's not necessary to remove your mask immediately after observing the green light. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

! WARNING: In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time.

USING YOUR SOCLEAN 2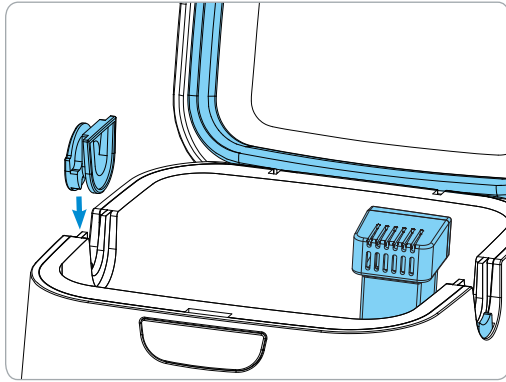

! WARNING: Do not physically alter the Hose Detect Switches on the SoClean 2. Regularly inspect your SoClean 2 for physical damage, such as cracks or broken structure, damaged gaskets, and cracks in the sleep equipment hose. Discontinue use if damage is detected and contact SoClean for assistance.

1. Open the Lid.
2. Install the Removable Hose Slot Plug into one open slot. Orientate the Removable Hose Slot Plug as shown above. The round face of the Removable Hose Slot Plug is on the outside of the SoClean 2.

Note: Your mask and PAP Hose can insert from either side of the SoClean 2. To change sides, remove the Removable Hose Slot Plug by lifting it straight out, and insert it into the slot on the opposite side.

3. **Remove outer plastic wrapping and blue tape from the Cartridge Filter** and fully seat it in the Cartridge Filter Slot at the back right corner of the SoClean Chamber. Ensure proper installation by orienting the Cartridge Filter vents to face the front of the machine.
4. Insert your PAP mask into the SoClean Chamber, allowing its attached hose to rest in the open Hose Slot, opposite the Removable Hose Slot Plug.
5. Close the Lid with the hose placed in the open slot. The SoClean 2 will now automatically operate according to its preset time and duration.



Note: The SoClean 2 won't operate without the mask and hose present in the closed Chamber. Additionally, the SoClean 2 will not operate if the Removable Hose Slot Plug is missing or improperly seated.

! WARNING: Keep the SoClean Lid closed until the green status light illuminates (approximately 2 hours after completion of the Cycle).

It is necessary to let the mask remain in the closed SoClean Chamber for at least **2 hours and 10 minutes** to ensure that the ozone performs its function and dissipates appropriately. After that period, the status light shines green and the mask is ready to be removed from the SoClean Chamber. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

Ozone quickly dissipates in the atmosphere. If you accidentally interrupt the Cycle and release ozone by opening the Lid, simply step away from the SoClean **for a minimum of 6 hours** to avoid unintended exposure.

Note: Your mask and hose will not be fully maintained if you interrupt the Cycle; therefore, it is recommended that you restart the Cycle.

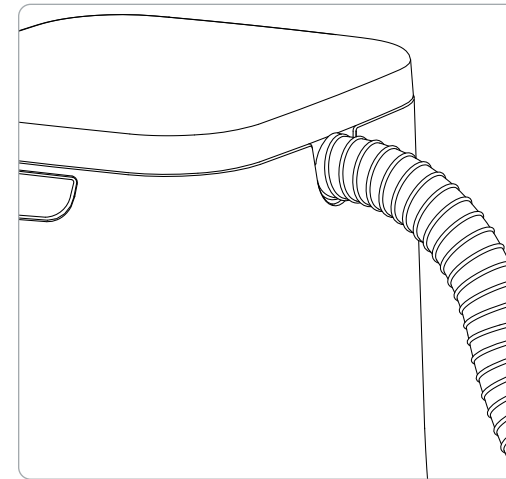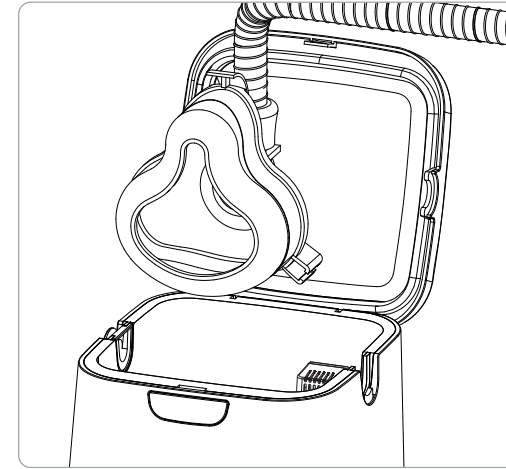
6. Once the cycle has completed (status light will be green), open the Lid and remove the mask.
7. Remove the Removable Hose Slot Plug and place inside the SoClean Chamber. Run your PAP for 1-2 minutes before putting your mask on.

Tip: Placing the Removable Hose Slot Plug into the SoClean Chamber helps prevent accidental loss of the Removable Hose Slot Plug.

! WARNING: Do not route your PAP Hose through both Hose Detect Slots while or after using your PAP. This may result in the SoClean 2 running at an unexpected time.


! WARNING: Only rest the flexible section of the PAP Hose, not the rigid connector in the Hose Detect Slot.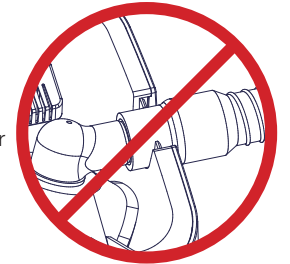


RECONNECTING YOUR PAP EQUIPMENT

Once the Maintenance Cycle is complete and you are ready to use your PAP hose and mask, wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.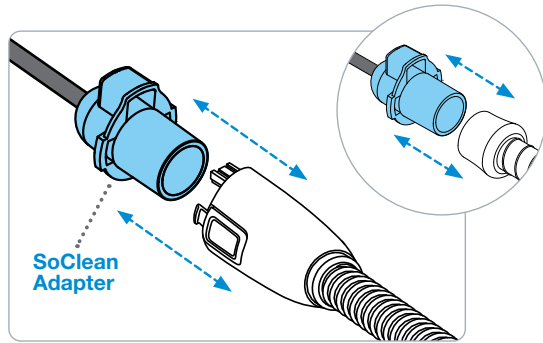

1. Remove the SoClean Adapter from your hose.

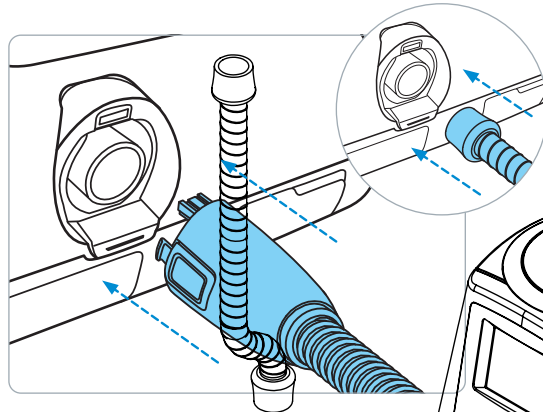2. Reconnect your hose to the PAP machine hose port.

CARE AND MAINTENANCE OF YOUR SOCLEAN 2 DEVICE

Wipe down the SoClean 2 with a damp cloth.

! WARNING: Do not submerge or fill the SoClean 2 device with water!

The SoClean Cartridge Filter Kit should be replaced every 6 months. A reminder message on the display will appear after approximately 6 months, depending on usage.

To reorder supplies (Cartridge Filter Kit), contact your authorized reseller or visit **SoClean.com** and search for the desired part or contact customer care.

Please remember to reset your SoClean 2 after installing the new Cartridge Filter as described in the instructions that come with the replacement.

Please purchase your Cartridge Filter Kits from SoClean® only. Authentic Cartridge Filter Kits can be purchased from SoClean via multiple online marketplaces, such as Amazon® in addition to SoClean.com. However, please confirm that the seller of your Cartridge Filter Kit is SoClean, and not any other seller, to ensure the authenticity of your Cartridge Filter Kit. SoClean has discovered counterfeit Cartridge Filter Kits being sold, particularly on Amazon® and eBay®, with Check Valves and Filters that leak and can cause damage to your SoClean machine or to your property due to water leakage from the counterfeit Check Valves. Please contact SoClean immediately if you have had any issues with your Cartridge Filter Kit.

Amazon® is a registered trademark owned by Amazon Technologies, Inc. eBay® is a registered trademark owned by eBay, Inc. SoClean® is a registered trademark owned by SoClean, Inc.

FAQS

1. Will my PAP equipment be wet from the SoClean 2?

No. No fluids or water are used in this process. If you observe water, please contact SoClean customer care at 800-916-4577.

2. Is the SoClean 2 harmful to me or the environment?

No. SoClean 2 produces ozone in a closed system and includes a Cartridge Filter which helps convert ozone back to regular oxygen. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

3. How do I know that my SoClean 2 is operating properly? After the cycle has successfully completed, the status indicator light will illuminate green.

Your PAP equipment will have a light, clean scent. If you have questions about the operation of your SoClean 2, please contact SoClean customer care at 800-916-4577.

4. What if the scent the SoClean 2 leaves is too strong for me? If you smell ozone or any other strong odor, air out the mask and hose by:

- Running your PAP equipment for 1-2 minutes before putting your mask on

If the device still produces a strong odor, try the following:

- Wash your PAP mask and hose based on manufacturers instructions.
- Use a mask wipe to clean your mask before putting it on.
- Make sure the Cartridge Filter is present, the clear plastic wrapping and blue sticker have been removed from the Cartridge Filter and that the black gasket is rolled up under the lip. Finally, make sure the Cartridge Filter is oriented correctly in the slot.

5. What if my indicator light is shining red or yellow?

This means that the Cycle is not yet complete. Do not open the Lid until the light turns green at which point you can remove your PAP mask and hose from the SoClean Chamber. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

6. It has been more than two hours since the SoClean 2 completed a Cycle, but the status light is still yellow.

Why? Some types of indoor lighting make it difficult to distinguish the yellow and green lights on the SoClean 2. Confirm that the green light is visible before opening your SoClean 2 to ensure that the Cycle is complete, and your PAP mask and hose are ready for use. Wait a minimum of 8 hours from the start of the Maintenance Cycle and use of the PAP mask and hose.

7. What if the indicator light is off after a scheduled cycle? The SoClean 2 light is green when a cycle completes successfully. If there is no light, the cycle did not start.

Common reasons for this are that the SoClean 2 is not powered, the Lid was not securely closed, there was no PAP equipment in the SoClean Chamber at the scheduled Daily Start Time or the Removable Hose Slot Plug is missing or not properly seated. If everything looks good, then close the Lid and press the **MANUAL** button to test. Any error message will appear on-screen, Please call customer care if you need further assistance.

8. What if I missed my scheduled Daily Start Time, but still wish to run it? You can use the Manual function to run a Cycle as long as you still have 8 hours remaining before you go to sleep.

9. Why does my display say Order Filter Kit?

This message appears approximately every 6 months, depending upon usage, as a reminder to regularly replace the Cartridge Filter Kit. To reorder, contact your authorized reseller or visit SoClean.com.

When you're ready to replace the Cartridge Filter Kit, follow the instructions that come with it and ensure you do the following:

- Make sure you remove the clear plastic wrapper and blue tape from the Cartridge Filter.
- Reset the Filter countdown by pressing the manual and hourglass buttons at the same time for about 3-5 seconds. You will see a smiley face on the display once it has reset, which confirms that process is complete.

10. Are there materials that I should avoid putting into the SoClean 2?

Do not use incompatible materials in the SoClean 2 such as memory foam, natural rubber, or nylon. Headgear often contains incompatible materials for your SoClean 2. Remove headgear before placing mask in the Maintenance Chamber.

11. My mask feels oily, what should I do? Skin can leave oil behind on your PAP mask and hose so you should always hand-wash your equipment as recommended by the manufacturer. You may wipe your mask with SoClean's Mask Wipes.

12. How often should I wash my PAP mask and hose?

Follow your PAP manufacturer's instructions for cleaning your PAP hose and mask with mild, unscented soap and warm, drinking-quality water.

13. If I lose power, do I need to reset my SoClean 2?

In the event of loss of power to your home or the SoClean 2, ensure the correct Clock time and Daily Start Time are set on the SoClean 2. Failure to do so may result in the SoClean 2 operating at an unexpected time. See instructions on how to set your Clock and Daily Start Time. If you need assistance with the reset, please call SoClean customer care at 800-916-4577.

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Copyright © 2011-2025
SoClean® is a registered trademark of SoClean, Inc.
1 Vose Farm Road, Peterborough, NH U.S.A. 03458
SoCleanTM est une marque déposée de SoClean, Inc.