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SoClean Adapter
for Somnetics™ Transcend™ and HDM Z1™

INSTRUCTIONS FOR USE

If you need help connecting
your adapter, contact SoClean®.


 EU 00800 5558 8811 | US 866-501-3705 | CA 800-916-4577

soclean.com | info@soclean.com


12 Vose Farm Road | Peterborough, New Hampshire 03458 USA

C000365 Rev A 0419


Connecting the Adapter
to the Somnetics Transcend and HDM Z1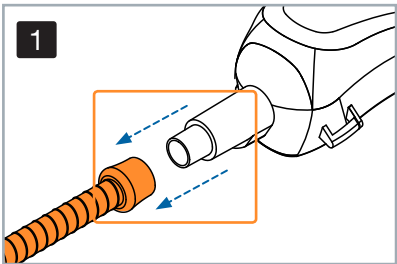


1. Remove the PAP hose from your Somnetics Transcend or HDM Z1.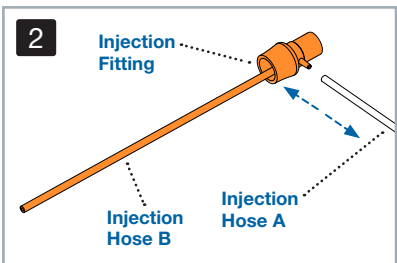

2. Detach the **Injection Hose A** from the **Injection Fitting** and **Injection Hose B**. Save the **Injection Fitting** and **Injection Hose B** in case you need them in the future.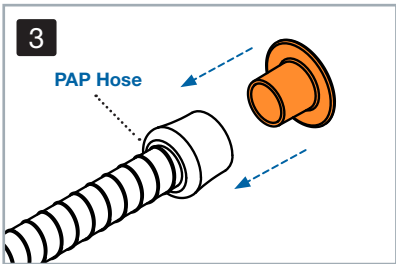

3. Install the SoClean Adapter onto the PAP hose.

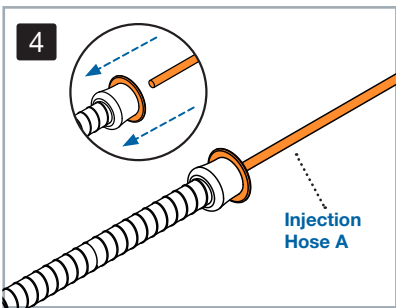
4. Connect **Injection Hose A** to the SoClean Adapter. You are now ready to use your SoClean.
- Place mask into the **SoClean Chamber** or **SoClean 2 Go Sanitizing Bag** and run the disinfecting cycle.


WARNING: **Injection Hose A** must be attached to the SoClean Adapter and the back of the SoClean device at all times when operating your SoClean.

Disconnecting the Adapter
from the Somnetics Transcend and HDM Z1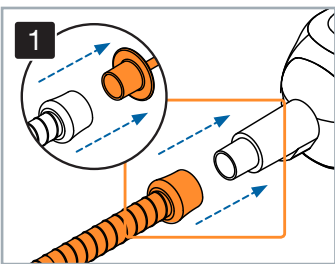

1. Once the disinfecting cycle is complete, please remove the SoClean Adapter from your PAP hose.

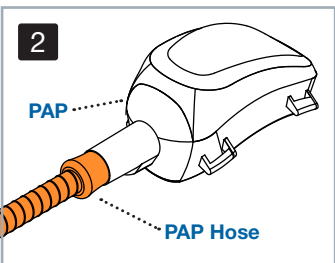2. Reconnect your PAP hose to your Somnetics Transcend or HDM Z1.
- You are now ready to use your PAP device.

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